



Flexible.  
Comfortable.  
Minimal.

The ResMed AirFit™ F40




Live life in high Res

**AirFit™**  
**F40**



# Help patients live a brighter life with better sleep

The ResMed AirFit™ F40 builds on what you and your patients love about ResMed masks. Featuring a fully flexible frame with a soft, total silicone AdaptiSeal™ cushion and fewer adjustment points, the AirFit F40 is designed for easy fitting and comfort. Its ultra-compact under-the-nose design and adaptive seal means patients can enjoy sleeping in any position.

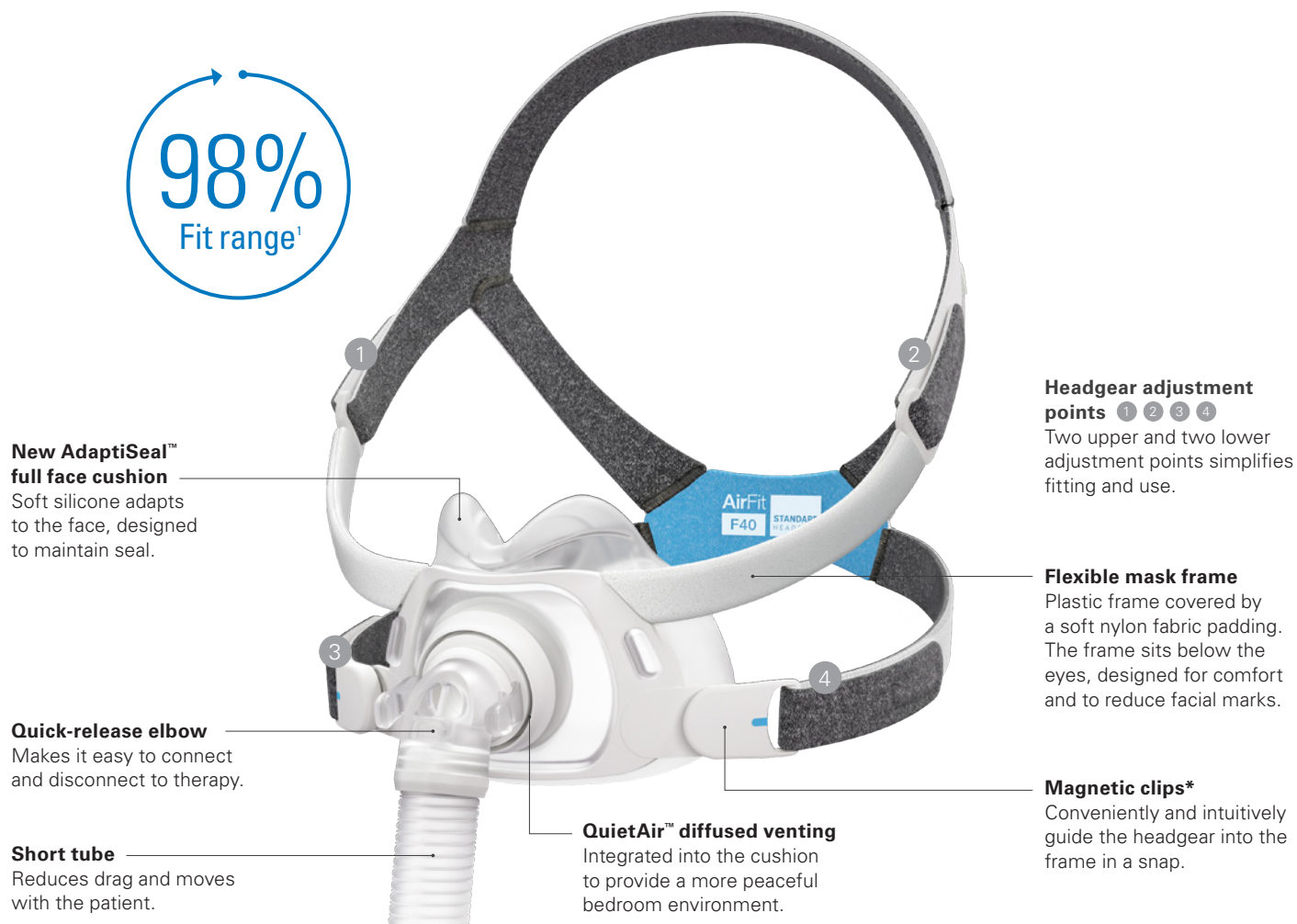

\*This mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.





## AirFit™ F40

### Complete mask system

Product name	Product code
AirFit F40 System – SW/STD APAC	64645
AirFit F40 System – M/STD APAC	64646
AirFit F40 System – L/STD APAC	64647

### Frame and vent ring

Product name	Product code
AirFit F40 Quiet Vent	64639
AirFit F40 Frame	64641

### Full face cushion

Product name	Product code
AirFit F40 Cushion Small Wide	64630
AirFit F40 Cushion Medium	64631
AirFit F40 Cushion Large	64632

### Headgear

Product name	Product code
AirFit F40 Headgear SML	64633
AirFit F40 Headgear STD	64634
AirFit F40 Headgear LGE	64635

Small in size. Big on features.

## How to fit the AirFit F40



1  
Unclip the lower straps away from the frame and place cushion under the nose to fit against the face.

With the ResMed logo facing up, pull headgear over head.



2  
Bring the lower straps behind the ears and reattach to the frame with the magnetic clips\*.

Feed longer hair through the back of the frame, if needed.



3  
Adjust the upper straps and then the lower straps for a snug fit.

## Experience the ResMed AirFit F40 today.

The minimalist ultra-compact full face mask is designed to deliver a reliable and adaptable seal. The soft, total silicone cushion moulds to the face and moves with patients as they sleep.

**The ResMed AirFit F40. A minimalist mask, that adapts to your patient's natural sleep.**

For more information, visit [ResMed.com/AirFitF40](https://ResMed.com/AirFitF40) or contact your ResMed sales representative.



\*This mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

<sup>1</sup> 97.7% Fit range based on data from two ResMed external clinical studies of experienced PAP patients. The first study included users with  $\geq 6$  months' therapy who used ResMed's AirFit F40 and AirFit F20 full face masks in randomized order for up to seven consecutive nights each at home. Study conducted in Australia, June 6 – 27, 2023, n=28 (AirFit F40) and 29 (AirFit F20). Facial measurements of 29 participants were taken. The second study included current users with  $\geq 12$  months' PAP therapy who met 90-day CMS compliance for PAP therapy in the three months prior to the study. Participants used ResMed's AirFit F40 and Fisher & Paykel's Evora Full in a randomized order with their own for up to seven consecutive nights each at home. Study conducted in the U.S., October 4 – November 20, 2023, n=57. Facial measurements of 57 participants were taken.

**ALWAYS READ THE LABEL AND FOLLOW THE INSTRUCTIONS FOR USE. You should speak to your doctor about your symptoms and whether a CPAP device is suitable for you.**



**ResMed Pty Ltd**  
1 Elizabeth Macarthur Drive  
Bella Vista NSW 2153 Australia

AirFit, AdaptiSeal and QuietAir are trademarks of the ResMed family of companies. For patent and other intellectual property information, see [ResMed.com/ip](https://ResMed.com/ip). © 2024 ResMed Pty Ltd. 10114179/1 2024-07